



## ***Installation Instructions for the RC5300W and RC5300R AED Cabinet***

### **RC5300W (Surface Mount) Cabinet Mounting:**


- 1) Locate studs or identify structural material capable of providing sufficient support.
- 2) Select fastener type (not included):
  - a) Steel studs—#10 x 2  $\frac{1}{2}$ " fine thread
  - b) Wood studs—#10 x 2  $\frac{1}{2}$ " coarse thread
  - c) Block or concrete—plastic wall anchors of minimum 1  $\frac{1}{2}$ " depth
  - d) Hollow wall—minimum grip of 5/8 inch
- 3) There are 4 mounting holes in the cabinet back intended for cabinet mounting. It is recommended that fasteners are utilized in all 4 holes. Position the cabinet such that a minimum of 2 fasteners attach to studs or concrete.
- 4) Fasten the cabinet to the wall.
- 5) Ensure door opens freely.

### **RC5300R (Recessed Mount) Cabinet Mounting:**

- 1) Prepare a rough opening of 17  $\frac{1}{4}$  inches high by 16  $\frac{3}{8}$  inches wide.
- 2) Place the cabinet inside the rough opening and level.
- 3) There are 2 mounting holes on each cabinet side intended for cabinet mounting. Secure with four #10 screws. (not included)
- 4) Ensure door opens freely.

### **Activate the Alarm:**

- 1) Insert the key into the activation switch mounted in the cabinet.
- 2) Turn the key to the unarmed position as shown in Figure 1.
- 3) Install the 9 volt battery into the holder mounted on the under side of the cabinet top.
- 4) Connect the 9 volt battery clip to the battery.
- 5) Close the cabinet door.
- 6) Turn the key to the armed position as shown in Figure 1.
- 7) Opening the door will trigger the alarm. Closing the door will stop the alarm.

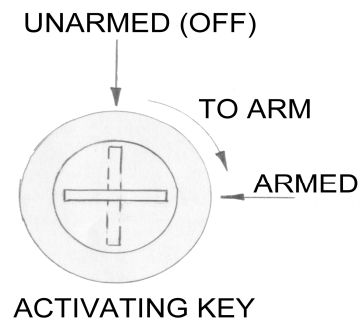

***Figure 1***

### **Maintenance:**

Replace the battery yearly or as needed.

U.S. Patents 6,301,501 / 6,735,473  
Canadian Patent CA 2,406,541

### **Warranty:**

One year limited warranty.

***HeartStation***  
***1000 Brown Street, Wauconda, IL 60084***  
***(800) 818-2939***