




# HeartCase Stow-n-Go™ (HC1) Installation Instructions

Accommodates Philips, Physio-Control, & HeartSine in soft case, or Welch Allyn without soft case

## Mounting the Bracket :

- 1) Locate studs or identify structural material capable of providing sufficient support.
- 2) Determine mounting height. This is dependent on the reach, direction of approach, and type of location. In an unobstructed location, a height of 44 - 50 inches from the floor is recommended (to the middle hole of the mounting bracket).
- 3) Select Fasteners (not included).
  - a) Steel studs—#10 x 2-1/2" fine thread
  - b) Wood studs—#10 x 2-1/2" coarse thread
  - c) Block or concrete—plastic wall anchors of minimum 1-1/2" depth
  - d) Hollow wall—minimum grip of 5/8"
- 4) Bracket has **2 mounting holes** with vertical spacing of 3 inches (top hole is for security tie). Mount the bracket with fasteners in both mounting holes.

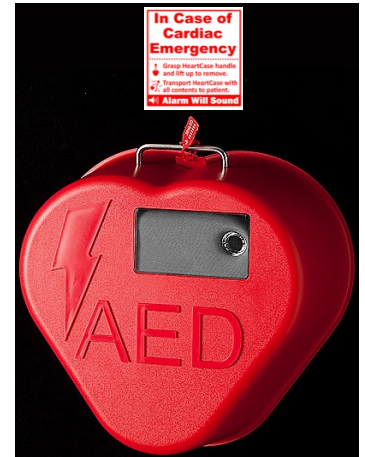

## Assembling the HeartCase Stow-n-Go™ & Installing AED:

- 1) Install handle on top of HeartCase™ with the two screws included.
- 2) Install hook & loop strap inside HeartCase™ (Diagram 1):
  - a) Place HeartCase™ on face with open side up.
  - b) Locate molded eyelets on the inside of the case. Thread ends of strap (**fuzzy side up**) from the bottom of the eyelets upward. Drape strap ends over sides of HeartCase™.
  - c) Slip the AED (face down) **UNDER ENTIRE** strap - be careful not to pull the strap out of eyelets.
  - d) Secure AED in place by pulling strap ends tight and securing against fuzzy side of strap.
  - e) Make certain the AED indicator lights are visible through the window.
- 3) Accessories such as a rescue kit can be placed under the strap or alongside the AED. Be sure they are strapped in place securely.

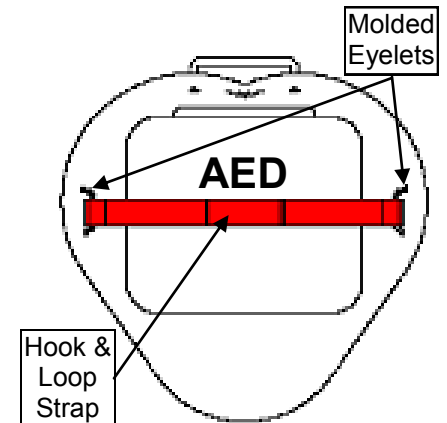


Diagram 1

## Hanging the HeartCase Stow-n-Go™:

- 1) If using the alarmed model, move alarm switch to the "On" position. (Alarm will sound but will stop when case is placed on bracket.)
- 2) Hang HeartCase™ (with all contents) on bracket. Bracket extends through the slot behind the handle (see photo above).
- 3) Mount the 6" X 6" HeartCase™ instruction sign on the wall above the top of the handle with mounting tape (included) or screws (not included) .
- 4) To install optional security tie see diagram 2. The security tie indicates use or tampering of the AED. It is easily broken to provide access.

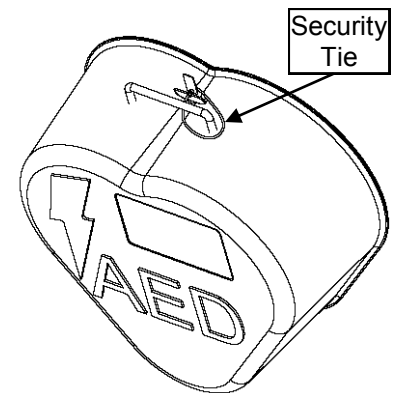


Diagram 2

**HeartCase is PATENT PENDING**  
HeartCase, Stow-n-Go, and heart shaped AED enclosures are trademarked by HeartStation affiliates.  
HeartStation has **EXCLUSIVE** rights to this pending patent and use of these trademarks.